Aukey

Ultra-Portable Video Magnifier

User Manual

Aumed
Seeing is Believing

English 01

Française 11

Italiano 23

Español 34

Deutsch 45

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Aukey
Ultra-Portable Video Magnifier
User Manual

English

Aumed®

Version Number_1.0 (English Version)

Copyright 2010 by Aumed,

All rights reserved

(Au100520)

Aumed[®]

Aumed Group Corp.

3/F Building D, No. 31 Jiaoda Dong Road,

Haidian District, Beijing 10044 China

Tel: 86-10-8288 4173 Fax: 86-10-8288 4170

Website: www.aumed.cn

Index

1. Introduction	2
About this manual.....	2
2. Safety and caution	2
3. What's in the box	4
4. Getting to know your Aukey	5
5. Getting started	6
Power on / off	6
Setting the magnification.....	6
Viewing modes	7
Image freeze	7
Detachable stand	8
Battery indicator	8
6. Specifications	8
7. Warranty	9

1. Introduction

Congratulations on choosing the Aukey, an ultra portable Video Magnifier from Aumed. Use the Aukey for reading, writing, viewing maps, menus and receipts, etc. We hope you enjoy working with your Aukey.

About this manual

This user manual provides a comprehensive guide to the features and operation of the Aukey. Pictures in this user manual may be different from the actual situation. Aumed reserves all the rights for the interpretation of all contents in this user manual. Please read this user manual carefully before using the Aukey and keep the user manual in a safe place for future reference.

2. Safety and Caution

- Do not expose the Aukey to extreme temperatures or humidity.

- Do not place Aukey near a radiator or heater, or where it will be exposed to direct sunlight. These conditions may damage the electronics or optics of the Aukey.
- Handle the Aukey with care. Rough handling will damage the internal components.
- Do not remove any parts of the Aukey. Contact your local Aumed distributor to service the Aukey if necessary.
- Do not open any of the components as this will void the warranty. Each component contains parts which should only be serviced by an Aumed repair specialist.
- Do not use cleaning agents or abrasive materials as they will damage your unit. Avoid touching the lens of the camera and keep it clean by using a soft cloth.
- Use only the power adapter provided.
- Do not hit, vibrate, squeeze or drop the equipment.
- Do not use the unit under +5°C (41F) or above +40°C (104F).

Use of the Aukey other than that described in this user manual will exclude it from warranty

conditions.

3. What's in the box?

The Aukey package contains the following:

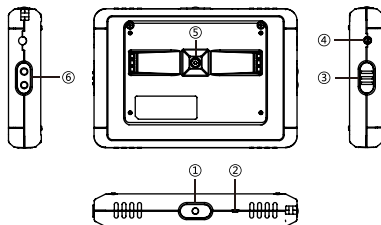
- ① The Aukey ultra portable video magnifier
- ② A power supply adapter
- ③ A wrist band
- ④ A detachable stand
- ⑤ A user manual
- ⑥ A carrying pouch

If any items are missing from your package, please contact your local Aumed distributor.

4. Getting to know your Aukey

The following figure shows the main components of the Aukey.

- ① Power on/off/ Viewing modes button;
- ② Power charging indicator;
- ③ Freeze/Unfreeze mode button;
- ④ DC in;
- ⑤ Centralized camera;
- ⑥ Zoom button;

5. Getting Started

Power On / Off

Power On: Press and hold button  for 4 seconds to turn on the device.

Power Off: Press and hold button  again for 4 seconds to turn off the device.

After powering on the Aukey, viewing mode is initially set to Full Color Mode. For optimal viewing keep the Aukey approximately 1" above the viewing object. Do NOT put the Aukey on the object directly.

Setting the magnification

There are 2 digital magnification levels by pressing the  button. The total combined optical and digital magnification ranges from 1.9X-13.6X magnification.

-Users can also change the distance between the camera lens and target object for getting intermediate degrees of optical magnification.

•Viewing modes

Press the button,  and select the different color modes on the LCD Screen. The Aukey has three different color modes available:

-Full Color Photo mode

This mode will display text, pictures and objects in full color.

-Negative reading mode

This mode reverses the positive image. Pictures and text will be displayed as white text on a black background.

-Positive reading mode

This mode enhances the foreground / background contrast. Pictures and text will be displayed as black text on a white background.

•Image Freeze

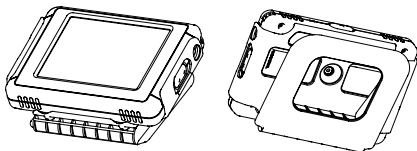
-Press  button to freeze the image for a closer look.

-Press the button again to exit freeze image mode.

- In image freeze mode, people can still change the color modes.

Detachable Stand

Use the detachable stand while reading for long time.

Detach the stand for travelling or shopping.

Battery Indicator

The LED beside the viewing modes button is the battery indicator.

Status of the indicator:

- Red light flashing means: the battery needs to be recharged;
- Green light flashing means: the battery is under recharging, once fully charged, the light will be turned off automatically.

6. Specifications

Magnification Range	1.9X-13.6X
Magnification Level	2 Levels
Viewing modes	3 color modes
Image Freeze Frame	Yes
Writing function	Yes
Li-ion Battery Capacity	1500mAh
Battery Working time	2.5 hours
Charging time	2.5-3 hours
LCD Screen size	3.5" TFT LCD
Device Dimension	105X72X17 mm 4.13X2.83X0.67 inch
Weight	120g (4Oz)
Power adapter	5V 1.5A

7. Warranty

Aumed offers one year Limited Warranty from the date of original purchase, please always keep your purchase receipt as it may be needed for any warranty claims.

- At our option, Aumed will repair or replace the device without charge if the problem is caused by defects in (Under Normal Use).

-Our warranty does NOT cover (other terms and conditions, Please refer to **Clause 2 Safety and Care**):

- Intentional damage or damage caused by excessive physical force is not covered by this warranty.
- Using device under extreme conditions (e.g. under excessive heat) or not following instructions (e.g. using an incorrect power adaptor) is also not covered under this warranty.
- Scratches and blemishes from normal use are considered normal wear and tear and are not covered by this warranty.

To expedite any warranty claim, please contact your local Aumed distributor with dated proof of purchase or email us at support@aukey.cn, your claims will be answered within 48 hours.

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